

Immunogenetics and Transplantation Laboratory

University of California, San Francisco

Principal Investigator

Lee Ann Baxter-Lowe, Ph.D. d(ABHI)

Primary Staff dedicated to the NMDP Donor-Recipient Pair Project

Victor Corpuz

SBT HLA-DPB1 High Resolution Typing Procedure

Last Revision February 2004

Genomic DNA is extracted from a 200 µl sample (i.e., frozen whole blood, PBLs, granulocytes or EBV transformed cell lines using a Qiagen QIAamp Blood Kit (Qiagen, Valencia, CA) according to the manufacturer's instructions. Polymerase chain reaction (PCR) methodology employing locus-specific intron primers for the HLA-DPB1 locus are used to generate templates for sequencing reactions. The size and quantity of the amplicons are determined using agarose gel electrophoresis.

Unincorporated primers and nucleotides are removed from amplicons using a High Pure PCR Product Purification Kit (Roche, Indianapolis, IN) or a PCR 96 Cleanup Kit (Millipore, Bedford, MA). The purified HLA-DP amplicons serve as templates for an exon 2 cycle sequencing reaction using Big-Dye sequencing chemistry (Applied Biosystems, Foster City, CA). Sequences are determined for the sense and anti-sense strands using nested primers or M13 priming sites created by the PCR primers. Unincorporated nucleotides are removed using Sephadex spin columns. Primer extension products from the cycle sequencing reactions are separated on an ABI Prism 3100 capillary sequencer (Applied Biosystems). If templates are heterozygous, the sense and anti-sense strands are sequenced.

HLA data are analyzed using ABI Sequence Navigator and ABI MatchTools software and a preliminary HLA type is assigned. If only one allele is observed, a low resolution sequence specific primer (SSP) assay is used to confirm homozygosity. Sequencing ambiguities are resolved using one or more of the following approaches: (1) sequencing of templates generated by allele-specific amplification, (2) typing using SSP methods (PelFreez, Milwaukee WI), and (3) sequencing of templates generated by cloning the locus-specific amplicons.

References:

Baxter-Lowe LA. (2000) Class II HLA Sequence Based Typing. *ASHI Laboratory Manual, 4th Edition*, H Noreen (ed), American Society of Histocompatibility and Immunogenetics, V.C.8.1

Hessner MJ Baxter-Lowe LA. Characterization of Three Novel HLA-DPB1 Alleles. *Tissue Antigens* 1992; **40**: 261-263.

Tilanus MGJ, ed. *IHWG Technical Manual: Genomic Analysis of the Human MHC: DNA-Based Typing for HLA Alleles and Linked Polymorphism* 2000: Publication of the 13th International Histocompatibility Working Group. Seattle, USA: Cancer Research Center.

Rozemuller EH, Eliaou JF, Baxter-Lowe LA, Charron D, Kronick M, Tilanus MGJ. An Evaluation of a Multicenter Study on HLA-DPB1 Typing Using Solid-Phase Taq-cycle Sequencing Chemistry. *Tissue Antigens* 1995; **46**:96-103.